### UPOFIX

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for non-precious, PD-, precious metal and reduced alloys as well as titanium

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for ceramic and extra-hard veneer materials (Isosit, Dentacolor, Elcebond, Visiogem)

2.000 - 5.000 min<sup>-1</sup>

Abverkauf clearance sale

REF	UP239	UP249	UP259
Farbkod/color code			
	10/5/1	10/5/1	10/5/1

### Recommendation for use

- UPOFIX receive all incorporated structures.
- Works on flat surfaces, but achieves also every niche. A possible application is vestibular and occlusal.
- The kevlar fibres with different special grits conform the latest technical knowledges..
- The best results will be reached by speeds os 2.000 - 5.000 rpm. Advisable for good results is an intermittent operation with moderate strong pressure and simultaneous circular, diagonally leading moves.
- Suitable for almost all dental material, including titanium.
- UP260: for all metals
- UP219: occlusal surface and interdental areas, metal pre-polisher
- UP229: occlusal surface and interdental areas, metal-high-shine-polisher
- UP239: ceramic as pre-polisher
- UP249: shine-polisher for ceramic, pre-polisher for composite, acryl
- UP259: shine-polisher for composite and acryl

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### Recommendation for use

Light = hard bristles  
Goat hair for pre-polishing  
Dark = soft bristles  
Natural hair (Chunking) for high-shine polishing

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- High-Shine Polishing of plastics
- optimal usage with polisher paste without adhesive
- long service life