CBR1S

REF	CBR1S
Ø (1/10 mm)	016
L (mm)	5,0
RA	016
	10

30.000 min⁻¹

CBR1V - Vollhartmetall full tungsten carbide

REF	CBR1V
Ø (1/10 mm)	018
L (mm)	8,0
RA	018
	10

30.000 min⁻¹

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Recommendations for use – C22 ...

- *designed for removal of bracket adhesive after debonding*
- *deliverable as FG and RA shank.*
- *The special toothing removes adhesive without causing damage to the tooth.*
- *The security chamfer at the toothing end reduces the danger of striae at the enamel surface.*
- *Different lengths of toothing facilitate the preparation of small and large areas.*
- *The smooth non-cutting tip assures gentle finishing without damage to the gingiva.*

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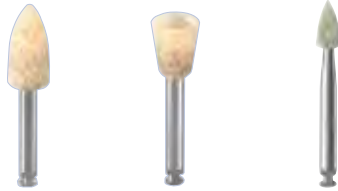
Recommendation for use

Safe removal of remnants without damage of the natural tooth substance.

Bond remover is an economical and safe alternative in contrast to conventional elimination of adhesive or cement residues with diamond or carbide instruments.

While removal with diamond or carbide instruments bears the risk to injure the natural tooth substance, with the bond remover this is excluded due to hardness of the included abrasive grain. The micro grit behaves defensively and is embedded in a hard synthetic caoutchuc matrix.

Safe and economical removal of remnants is possible in few minutes.

Please cool and work with low speed.