### ökoCleaner

REF	OEC2501	OEC2502	OEC2504	OEC2505	OEC2506
REF Sortiment / assortment			OEC2500		
Schaft / shank	RA	RA	RA	RA	RA
	10	10	10	10	10

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- gentle removal of hard and soft plaque
- for finishing of filling and preparation
- allows the subgingival root smoothing and polish of difficult achieved interproximal root areas
- elimination of cement excesses without damage of neighbouring soft tissues in the presentation of temporary restoration or implants
- removal of remnants
- root smoothing after scaling

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### Properties

- instruments are self-sharpening by continuous abrasion
- no abrasion of enamel and ceramics
- gentle treatment of dentin, root cement and composite
- instrument slides over soft tissue

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### Application

- for the use with contra angle handpiece with reduced speed rotation
- working speed rotation: 10.000 - 15.000 rpm
- please work with using slight pressure and water spray
- cleaning in medical alcohol
- sterilising in the autoclave up to 135°C

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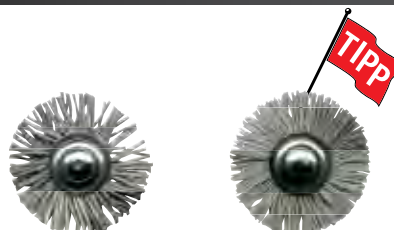
### Practical advice

- pointed shape preferred for polishing in down approximal areas
- paro figures are suitable especially for polishing of furcations at the molars

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For ceramic and extra hard veneer resin (iosit, dentacolor, Elcebond, Visiogem).

REF	UP249	UP259
Ø / L (in mm)	220	220
Schaft / shank	HP	HP
	10/5/1	10/5/1

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### Recommendation for use

- UPOFIX receives all incorporated structures.
- The best results are achieved with speed 2.000 - 5.000 min<sup>-1</sup>.
- Smoothing of occlusal and bite splint
- Careful elimination of calculus on prothesis after ultrasonic cleaning
- Finishing of plastic temporary restoration.

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### Recommendation for use

**PROPHY-WHITE** and **BLUE** are especially efficient because the polishing paste is incorporated in the material. They are suitable for recall after orthodontic and ultrasonic treatment.

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- *removes discoloration of all kinds*
- *applicable with and without polishing paste*
- *disposable product*
- *latex-free*

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*For teeth cleaning (sterilized in the autoclave up to 145 °C)*

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### Recommendation for use

**PROPHY-BRUSH** is hygienically harmless, because for using are only concluded nylon fibres - no capillary action.

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### Application possibilities

- *Prophylaxis (cleaning the approximal area)*
- *Cleaning of implants*
- *Cleaning of brackets*
- *Cleaning of the root canal*
- *Suitable for utilization under bridges*

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*Polisher with snap-on mandrel for application with polishing paste*

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### Recommendation for use

**PROPHY-SNAP** is very economical because of very durable mandrel and long usable chalices.

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